80330 Desinfektionsmittel-Halter

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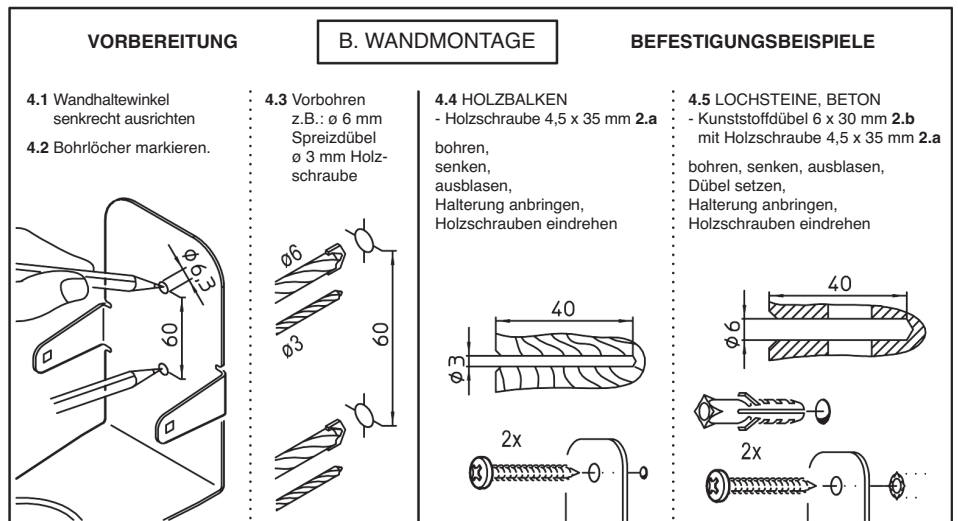
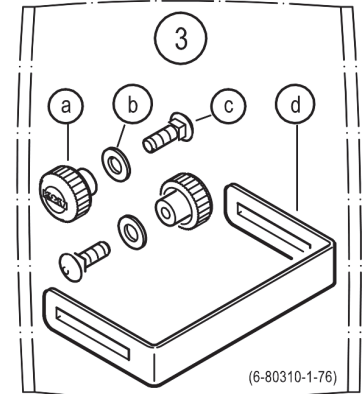
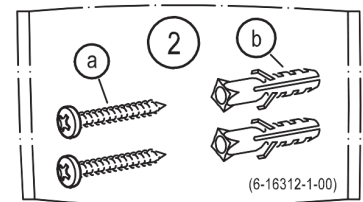
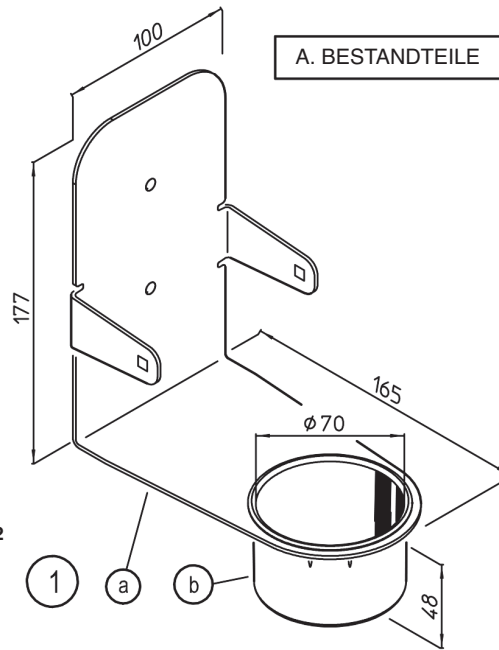
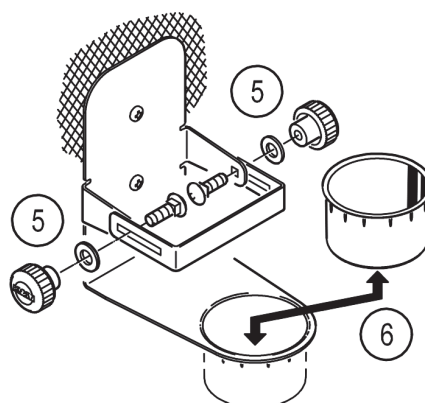
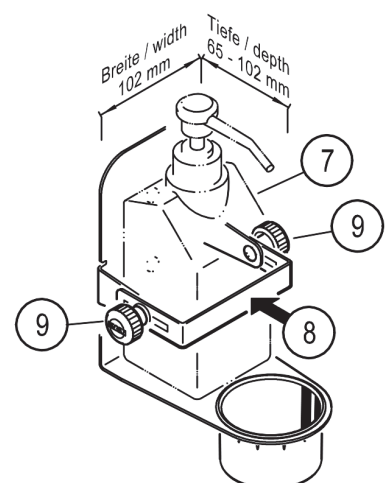
D. DESINFEKTIONSFLASCHE

80330 Disinfectant holder

Please read these assembly instructions carefully!

TECHNICAL DATA

DIMENSIONS

- Bottle size: width x depth: max. \varnothing 102 mm
- Clamping range: depth 65 - 102 mm
- Drip cup: \varnothing 70 x 47 mm

MATERIAL

- Mounting bracket: Steel
- Drip cup: Polyamide
- Knurled knobs: Polyamide, steel, zinc
- Screwing material: Steel, brass, zinc

SURFACE

pure white, powder-coated

WEIGHT

0.6 kg

SAFETY INSTRUCTIONS

The safety of the stand depends on the:

MOUNTING

Ensure that all screw connection are tight.

WALL CONDITION

Only use the product on suitable walls, i.e. mount only on load-bearing walls. Weak walls are unsuited as well as wall areas where running wires and pipes are hidden behind. The screws and dowels 2 supplied are not convenient for all walls. In case of any doubt consult a qualified expert.

LOAD

Only for disinfectant bottles with suitable size; see above TECHNICAL DATA.

SURROUNDINGS

Please avoid respectively prevent negative external influences (knocks, dragging, vibrations, wind, humidity etc.)

INSTALLATION INSTRUCTIONS

A. COMPONENTS

1 Wall holder

- 1.a Mounting bracket
- 1.b Drip cup (clipped in)

2 Accessories bag (6-16312-1-00)

- 2.a Wood screw 4,5 x 35 mm (2x)
- 2.b Anchor (plastic) \varnothing 6 x 30 mm (2x)

3 Accessories bag (6-80310-1-76)

- 3.a Knurled knob M5 (2x)
- 3.b Washer \varnothing 5,3 mm (2x)
- 3.c Carriage bolt M5 x 12 mm (2x)
- 3.d Clamping bracket

B. WALL MOUNTING

4 see fig. 4.1 - 4.5

C. PREPARING DISINFECTANT HOLDER

5 Open the accessories bag 3:

Fitting the clamping bracket 3.d to the wall holder 1:

- Slide the bracket 3.d from outside over the mounting bracket,
- insert the carriage bolts 3.c from the inside,
- thrust washers 3.b over the threads,
- screw on knurled knobs 3.a.

6 The drip cup 1.d can be removed for cleaning purposes.

Subsequently press the drip cup from above back into the holder.

D. DISINFECTANT BOTTLE (not included in delivery)

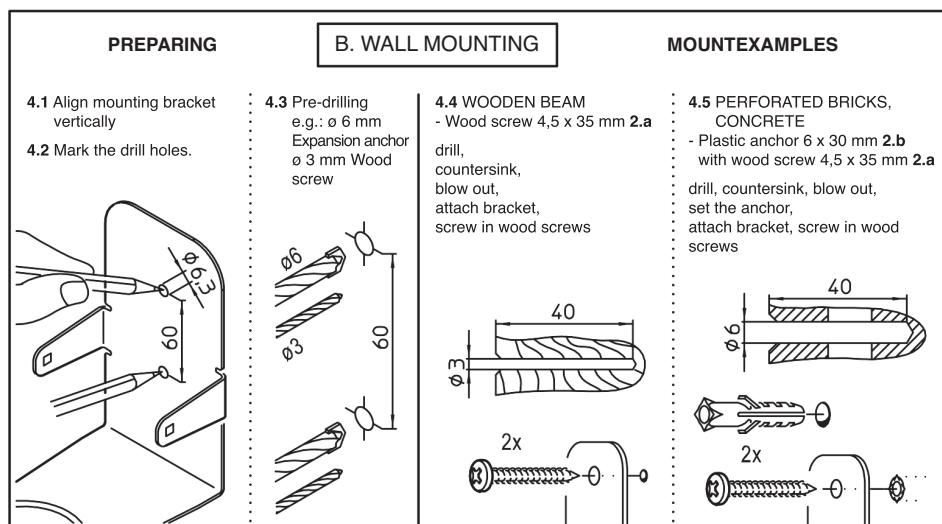
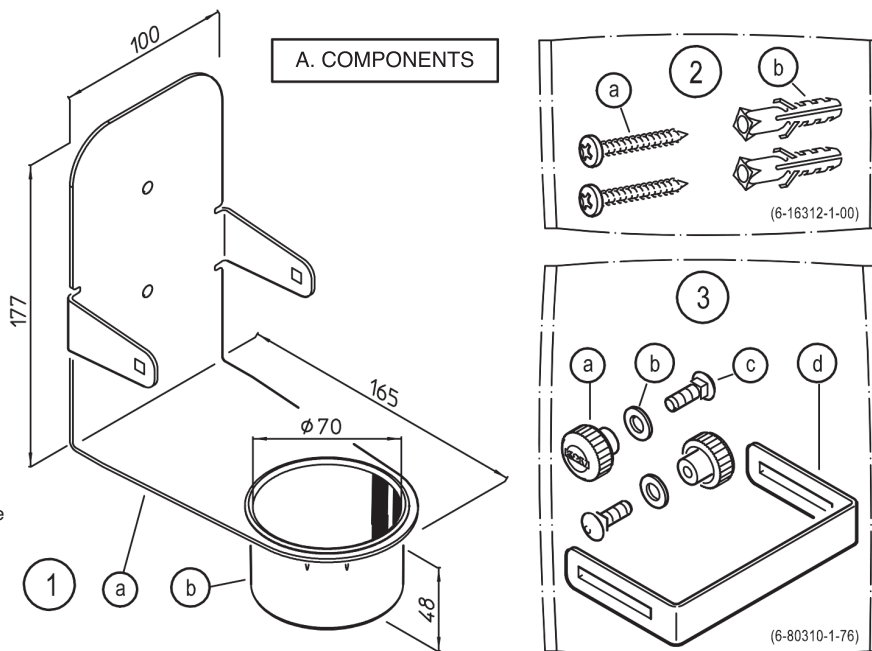
- 7 Place the disinfectant bottle in the bottle holder,
- 8 clamp the bottle by means of the bracket 3.d,
- 9 tighten both knurled knobs 3.a.

INFORMATION


NOTE: The disinfectant holder is not a medically certified product.

- Negative effects of the disinfectant on the products (surfaces, material) are not known to us.
- Nevertheless, we recommend to always remove possible residues of the disinfectant in order to avoid possible damage and to enjoy the product as long as possible.
- For cleaning purposes use a slightly damp cloth and a non-abrasive cleaning agent.

Disinfectant is not included in the scope of delivery.

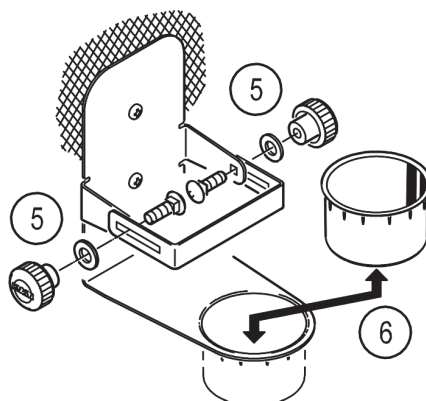C. PREPARING HOLDER

D. DISINFECTANT BOTTLE